


EPIPEN[®]


(Epinephrine) Auto-Injectors 0.3/0.15mg

userguide

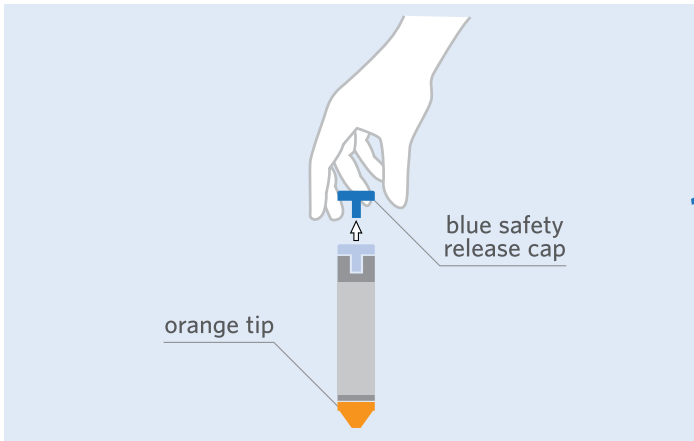

1

Pull off the blue safety release cap.

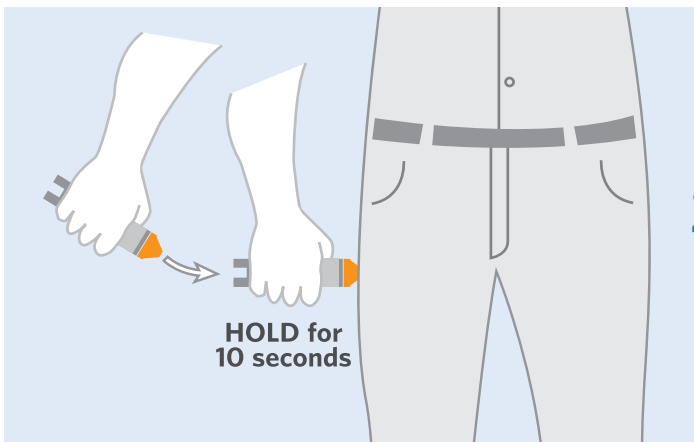

2

Swing and firmly push the orange tip against the outer thigh so it 'clicks.' HOLD on thigh for approximately 10 seconds to deliver the drug.

Please note: As soon as you release pressure from the thigh, the protective cover will extend.

Each EpiPen Auto-Injector contains a single dose of a medicine called epinephrine, which you inject into your outer thigh. **DO NOT INJECT INTRAVENOUSLY. DO NOT INJECT INTO YOUR BUTTOCK,** as this may not be effective for a severe allergic reaction. In case of accidental injection, please seek immediate medical treatment.

Call 911

3

Seek immediate emergency medical attention and be sure to take the EpiPen Auto-Injector with you to the emergency room.

To view an instructional video demonstrating how to use an EpiPen Auto-Injector, please visit epipen.com.

©2010 Dey Pharma, L.P. All rights reserved.

DEY[®] and the Dey logo are registered trademarks of Dey Pharma, L.P.

EpiPen[®], EpiPen 2-Pak[®], and EpiPen Jr 2-Pak[®] are registered trademarks of Mylan Inc. licensed exclusively to its wholly-owned subsidiary, Dey Pharma, L.P.